

Part Number	Pi14.IPL3
Description	Pi14.IPL3 Green Polycarbonate Lens
Product Picture	
Materials	Polycarbonate
VLT	9%
CE Laser Etched Specifications	na
OD Laser Etched Specifications	300-1400nm (OD 1) 9000-11000nm (OD 7)
Instructions for Cleaning and Disinfection	Use the enclosed cleaning cloth and mild soapy water. Rinse eyewear thoroughly with clean warm water. Dry with the enclosed clean, soft cloth. Store the eyewear within the provided case, in a dry and cool place. Do not use abrasive chemicals or alcohol to clean or disinfect.
Instructions for Use	a) It is the user's responsibility to make sure that the eyewear is appropriate for the IPL source being used. If you have questions consult the IPL manufacturer or your LSO. b) Due to the tinted and colored filter, recognition of warning lights or warning signals can be impaired. c) For additional information regarding specific data, and declaration of conformity visit https://innovativeoptics.com/ipl-eyewear/ d) Do not modify this product without prior approval from the manufacturer. e) Do not clean eyewear with abrasive cloths or chemicals. f) We recommend use for a period of 5 years. The duration of use depends on various factors such as use, cleaning, storage and maintenance. Frequent inspections and replacement is recommended if damaged.
Warning	Materials that contact the wearer's skin may cause allergic reactions to susceptible individuals.
Revision Date	10.20.2025